

Surgical Tray Corner Protectors, Reusable- 2894M1

SUGGESTED INSTRUCTIONS FOR USE



DESCRIPTION/INTENDED USE:

Surgical Tray Corner Protectors, Reusable (2894M1) are designed to prevent the sterilization wrap from tearing and ripping due to sharp corners or feet on surgical trays. They eliminate the accidental corner ripping of sterile wrapping and reprocessing costs. They fit all types of corners, even the round ones. Sold non-sterile, the latex free Corner Protectors are marketed both in single use (blue) and reusable (orange) versions.

The reusable Corner Protectors bright orange color make them easy to identify against sterilization wrap and reduce the risk of being discarded

WARNING/PRECAUTIONS

- ⚠ Visually inspect the corner protectors before use to ensure that they are free from visible

contaminants. **INSTRUCTIONS FOR USE**

1. Place the Corner Protectors on the top and bottom corners of the tray before wrapping with sterilization wrap. Once in place, wrap the tray according to standard wrapping procedures.
2. **Cleaning and Sterilization :**
The reusable corner protectors can withstand repeated sterilization up to 134°C (273°F) and were tested up-to 10 steam sterilization cycles without any degradation. Although the product is designed for considerable re-use, variables such as frequency of sterilization, cleaning application and handling can impact the product's useful life.
 - The corner protectors do not need to be cleaned or disinfected prior to initial use.
 - Between usages, inspect for wear or damages. If observed, discard in compliance with your facility disposable instructions.
 - Prior to subsequent re-use, the corner protectors should be cleaned according to the established cleaning protocol in your facility and visually inspected as mentioned above.

PRODUCT DISPOSAL

There are no special disposal instructions, discard in compliance with your facility's disposable instructions.



Keep in dry area



CE Marked



Keep away from
direct sunlight



Non Sterile product