

## Disposable, Single Use, Surgical Tray Corner Protectors, Reusable- 2894M2

### SUGGESTED INSTRUCTIONS FOR USE


---



#### DESCRIPTION/INTENDED USE:

Surgical Tray Corner Protectors, Disposable (2894M2) are designed to prevent the sterilization wrap from tearing and ripping due to sharp corners or feet on surgical trays. They eliminate the accidental corner ripping of sterile wrapping and reprocessing costs. They fit all types of corners, even the round ones. Sold non-sterile, the latex free Corner Protectors are marketed both in single use (blue) and reusable (orange) versions.

#### WARNING/PRECAUTIONS

-  Although the single use product appears similar to the re-usable, please be advised that it is blended differently to reduce costs and has only been validated as single-use. Re-using the single use Corner Protectors may compromise sterility.

#### INSTRUCTIONS FOR USE

1. The Corner Protectors do not need to be cleaned or disinfected prior to usage
2. Place the Corner Protectors on the top and bottom corners of the tray before wrapping with sterilization wrap. Once in place, wrap the tray according to standard wrapping procedures.

#### CLEANING & STERILIZATION

Corner protectors can withstand sterilization up to 134°C (273°F)

#### PRODUCT DISPOSAL

There are no special disposal instructions, discard in compliance with your facility's disposable instructions.



Keep in dry area



Keep away from  
direct sunlight



Non Sterile product



CE Marked

