



Gel Mats Medical Mat Care Instructions

FAILURE TO FOLLOW CARE INSTRUCTIONS WILL VOID YOUR WARRANTY.

After unpacking your mat, you may notice small wrinkles on the outside edges of the mat and possibly some on the top surface. These are normal and should disappear within 48 hours of placing the mat on a flat surface. Your mat requires a minimum of care, but certain activities should be avoided to increase the usable life of the product.

For non-hospital environments, clean with water and a damp sponge or cloth. If you have a sticky or greasy spill you may use a mild soap (such as dish washing detergent) and water with a damp sponge or cloth. Do not use any other household cleansers such as, but not limited to, window cleaners, floor cleaners, polishes, or all-purpose cleaners.

For in-hospital environments, Gel Medical Mats may be cleaned with a low-level, hospital grade disinfectant cleaner which kills blood-borne pathogens. We recommend using 3M™ or Ecolab® HB Quat Disinfectant Cleaner, diluted and dispensed as prescribed in the cleaner's general use directions, or a cleaning agent similar in composition. The Gel Medical Mat can be Cart Wash cleaned using the plastic mode setting.

DO NOT clean with a power vacuum cleaner. The high speed rotating brush can damage the surface of the mat.

DO NOT fold the mat as this can cause permanent damage.

DO NOT use the mat in wet environments or step on the mat with wet feet. The Gel Medical Mat is not intended for use in wet areas. The surface of the mat may be slippery when your feet are wet.

DO NOT place furniture or equipment on the Gel Medical Mat. Permanent indentions on mat may occur.

DO NOT allow water/liquid to become trapped under the Gel Medical Mat. If any moisture/liquid becomes trapped under a Gel Medical Mat, it may not dry and, therefore, it should be removed as soon as possible. When mopping your floor, please ensure that your floor has completely dried before replacing the mat. Prolonged exposure to moisture/liquid trapped under the Gel Medical Mat may cause mold/mildew and/or damage to your mat and some types of flooring surfaces.

ALL AREA RUGS AND FLOOR MATS CREATE RAISED SURFACES WHEN PLACED ON YOUR FLOOR FOR USE. THE GEL MAT CREATES A RAISED STANDING SURFACE WITH ITS COMFORTABLE SOFT GEL LAYER. ANY RAISED SURFACE COULD BE A TRIP HAZARD, AND CARE SHOULD BE TAKEN WHILE STEPPING ON, OVER, OR OFF OF AREA RUGS AND FLOOR MATS.