



Veinlite Quick Guide

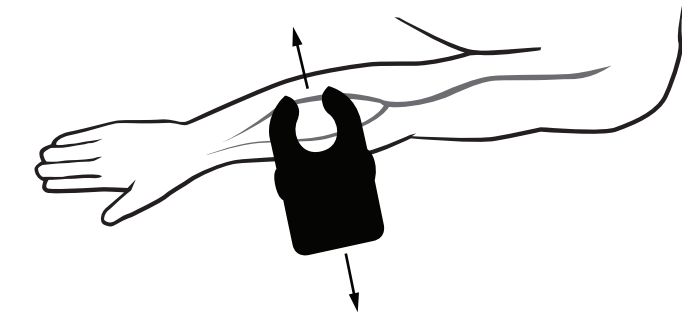

1

FIND THE VEIN


To scan for veins, place device **DIRECTLY** on the skin, perpendicular to vein direction. Scan side to side. Veins will be visible as dark lines due to transillumination

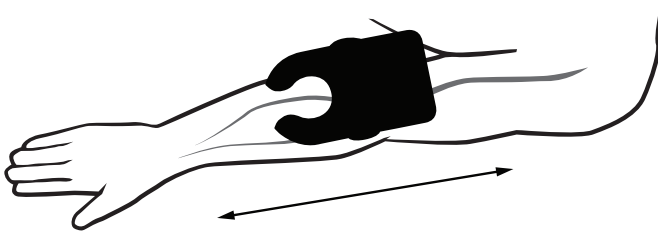

2

IDENTIFY THE VEIN


Once a suitable vein has been identified, rotate Veinlite 90 degrees, parallel to the vein.

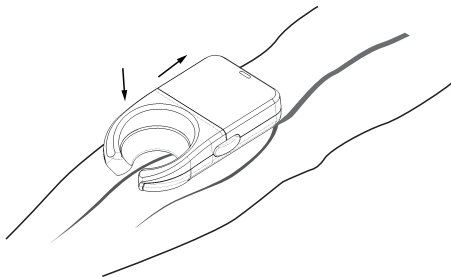

3

SECURE THE VEIN

gently apply pressure on the unit and push back to anchor the vein. This provides traction to the skin.

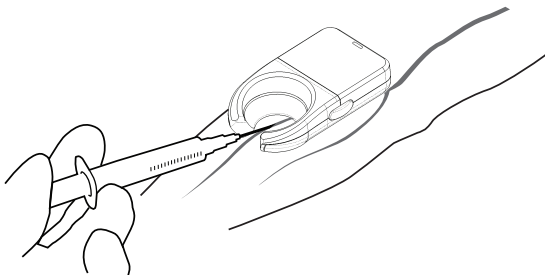

4

ACCESS THE VEIN

Insert the needle into the vein, through the opening of the Veinlite.