



### **SUGGESTED INSTRUCTIONS FOR USE**

## **2894M8 - Surgical Tray Corner Pocket Protectors, Reusable, Pink (80 Pack)**

(TEM NO.)	DESCRIPTION	UOM
2894M8	Pink Corner Pockets - Reusable	80 pcs/Bag

### **DESCRIPTION/INTENDED USE:**

Surgery Tray Corner Pocket Protectors are designed to prevent the sterilization wrap from tearing and ripping due to sharp corners or feet on surgical trays. They eliminate the accidental corner ripping of sterile wrapping and reprocessing costs. They fit all types of corners, even the round ones.

Sold non-sterile and latex free Corner Pockets. The reusable Corner Pockets bright pink color make them easy to identify against sterilization wrap and reduce the risk of being discarded.

### **WARNING/PRECAUTIONS**

- ⚠ Visually inspect the corner protectors before use to ensure that they are free from visible contaminants.
- ⚠ Requires replacing verification and/or validation and routine monitoring of the process.

### **INSTRUCTIONS FOR USE**

- Place the Corner Pockets on the top and bottom corners of the tray before wrapping with sterilization wrap. Once in place, wrap the tray according to standard wrapping procedures.
- Cleaning and Sterilization:**  
The reusable corner pockets can withstand repeated sterilization up to 134°C (273°F) and were tested up-to 10 steam sterilization cycles without any degradation. Although the product is designed for considerable re-use, variables such as frequency of sterilization, cleaning application and handling can impact the product's useful life.
  - The corner pockets do not need to be cleaned or disinfected prior to initial use.
  - Between usages, inspect for wear or damages. If observed, discard in compliance with your facility disposable instructions.
  - Prior to subsequent re-use, the corner protectors should be cleaned according to the established cleaning protocol in your facility and visually inspected as mentioned above.

### **CONTRAINDICATIONS**

No known contraindications and/or adverse effects.



#### **STORAGE**

Store in a manner that will reduce cross-contamination. Keep in dry place, ambient temperature and away from direct sunlight.

#### **PRODUCT DISPOSAL**

There are no special disposal instructions, discard in compliance with your facility's disposable instructions.

#### **LABEL SYMBOLS**



Keep in dry area



Keep away from direct sunlight



Non-Sterile product



Does not contain natural latex



CE Marked



Do not re-use



Indicates the item is a medical device



Manufacturing date



Caution, consult accompanying documents



Unique Device Identifier